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BeautyPad Pro 3.0

Instructions for use

BeautyPad Pro 3.0

Mode d'emploi

BeautyPad Pro 3.0

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BeautyPad Pro 3.0

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BeautyPad Pro 3.0

BeautyPad Pro 3.0

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BeautyPad Pro 3.0

This user manual lays out the intended use of this Permanent Make-up unit and serves to prevent malfunctions or damages to the unit. This unit may only be used by authorized medical personnel, beauticians as well as trained qualified personnel and only for the purposes mentioned in this user manual! All persons using, caring for or checking this unit have to read this user manual in all its details.

Application

The purpose of this Permanent Make-up unit is to insert colour into the skin by means of needles fitted into the hand piece. The control unit secures the electrical connection requirements for the hand piece.

Starting up the unit

The unit has been manufactured according to the VDE guidelines and will be delivered to the customers in perfect condition. An inspection according to the GS and VDE regulation has taken place. The unit may only be connected to a correctly installed power outlet. The voltage has to correspond to the input voltage stated on the type plate of the power adaptor!

The unit may only be opened for maintenance and repairs by the manufacturer or authorized personnel. The unit may only be used inside and can be operated without any problems at room temperatures between 10°C to 35°C.

In the case where operated the unit does not appear safe anymore it is to be switched off immediately and secured against unauthorized use. This will be the case if the unit:

- or its accessories show visible damage
- does not work correctly
- has been stored for a long time under adverse conditions
- was transported improperly or if it has been subjected to substantial mechanical stress (shock, crashes etc.).

The unit must always be positioned on a solid and level surface and may only be operated by trained personnel.

Control unit

First connect the power adaptor to the mains (100–240V) then connect the plug at the other end of the cable to the unit. You will find the connecting socket at the back of the unit. (POWER) Now connect the foot pedal and the hand piece to the appropriate and labelled connecting sockets.

Now an automatic device-test will start.

After activating the  sensor surface a value for the operating speed will appear in the 3-digit 7-segment display. This can be adjusted in steps of 10 on the sensor surface  and  respectively from 90 to 250.

The  sensor surface has to be touched in order to activate the hand piece. The LED light right above this button shows that the hand piece is now switched on. By touching this button again the hand piece will be switched off.

The hand piece can also be operated with the foot pedal if it has been connected to the "FOOT" connecting socket.

By touching the  surface you can switch between hand pieces if two hand pieces have been connected at the same time.

Attention! All marked areas to change functions are sensor surfaces and react to the slightest touch (not pressure).

When touching the button  the unit will change the Permanent Make-up application to the operation of the Plasma pen. This is identified by the symbol  on the display.

The connected Plasma pen (Type XXX) is now ready to use.

Hand piece

The hand piece for this unit has been designed for the use of modules.

The modules are being locked into the handle piece by means of clamping.

This takes place by pushing the module into the opening in the handle piece until it stops.

Hold on tight to the handle piece and turn the module clockwise (check if the needle sits tight by slightly pulling on it!).

Now the desired needle length can be set, by turning the handle piece. Please switch on the hand piece first.

To remove the module hold on tight to the handle piece and turn the module anticlockwise.

Remove the handle piece from the unit before cleaning the hand piece.

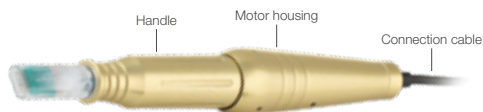
The handle piece is made off aluminium and can be cleaned by ultra sound or under running water. Add a small amount of a "mild" disinfectant (f.e. Lycetol Med; please pay attention to the applications instructions!) (do not use a undiluted concentrates). The remaining part of the hand piece can be cleaned with a damp cloth.

Attention! Do not submerge the hand piece in any liquid.


After cleaning it is recommended to apply a small amount of Vaseline on the threads of the hand piece.

The use of needles not recommended by the supplier will be at the users own risk.

Please avoid holding the hand piece with the tip pointing up. Risk of contamination!

Inserting/removing modules

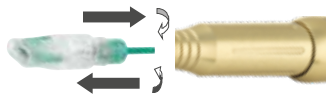Technical data

Plug-in power supply:	ZDD120100m-N
Mains input voltage:	100–240V AC/12V DC
Measurements (LxWxH in mm):	320x140x20 (70)
Weight:	approx. 1300g
Weight of hand piece:	approx. 100g
Operating speed of hand piece:	90–250
Type of needles:	Module
Motor in hand piece:	precision-DC-Motor
Switching output Plasma pen:	5V DC/0,3A
Operating mode:	Continuous operation
Operating conditions:	Ambient temperature 10°C–35°C Relative humidity 30%–75%

CE-Symbol

The unit complies to the conditions of the EMV-guideline 89/336 as well as to the low-voltage directive 73/23/EWG.

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Better Feet A/S / Cair Danmark